

Date/Time: 10/25/2018 9:34:25 AM
 Username: ADMIN

UDP Report

Instrument #: CT0271(2)
 Version #: V4.70 A

400027CMAX S4

Test Name	Version	Extraction Type	Master Mix Format					Ct Calculation	Lysis Heat Time
Notes	Channel Settings	475/520	530/565	585/630	630/665	680/715	Lysis Temperature	Sample Tip Height	
							Sample Volume	Wash Volume	
							Neutralization Volume	DNase Heat Time	
							Liquid Level Sensing Enabled		
							400027CMAX S4	1	ExK DNA-3 - 4-Snap
--		Alias	--	HSV2	HSV1	Hducr	SPC	62	1600
		Gain	--	60	60	60	60	500	500
		Threshold	--	200	200	200	200	--	--
		Excitation Channel 475/520	--	0	0	0	0		
		Excitation Channel 530/565	0	--	1	0	0		
		Excitation Channel 585/630	0	0	--	3.5	0		
		Excitation Channel 630/665	0	0	5	--	0		
		Excitation Channel 680/715	0	0	0	1	--		

Enabled

Step Name: New Step
Profile Type: Hold **Cycles:** 1

Type	Time (s)	Temp (°C)	Detect
--	300	99	No

Step Name: New Step
Profile Type: 3 - Temperature **Cycles:** 40

Type	Time (s)	Temp (°C)	Detect
--	11.5	99	No
--	32.5	58	Yes
--	20	72	No

	HSV2			
HSV2 - PCR	✓	✓	✗	✗
SPC - PCR	✓	✗	✓	✗
Results	➕ POS	➕ POS	➖ NEG	ⓘ UNR

	HSV1			
HSV1 - PCR	✓	✓	✗	✗
SPC - PCR	✓	✗	✓	✗
Results	➕ POS	➕ POS	➖ NEG	ⓘ UNR

	Hducr			
Hducr - PCR	✓	✓	✗	✗
SPC - PCR	✓	✗	✓	✗
Results	➕ POS	➕ POS	➖ NEG	ⓘ UNR



Performed By: _____ Date: _____
 Reviewed By: _____ Date: _____

Printed: 2018-10-25 09:34:26

Page: 1/1