

Date/Time: 6/26/2020 10:08:52 AM  
 Username: ADMIN

# UDP Report

Instrument #: CT1557(1)  
 Version #: V4.72 A

## 400058HMP T3

Test Name	Version	Extraction Type	Master Mix Format				Ct Calculation	Lysis Heat Time
Notes	Channel Settings	475/520	530/565	585/630	630/665	680/715	Lysis Temperature	Sample Tip Height
							Sample Volume	Wash Volume
							Neutralization Volume	DNase Heat Time
							Liquid Level Sensing Enabled	
							400058HMP T3	5 ExK TNA-3
	Alias	CoVN2	--	CoVN1	--	RNase P	60	1600
	Gain	80	--	80	--	60	937.5	500
	Threshold	150	--	150	--	200	--	--
--	Ct Min	--	--	--	--	--	--	--
	Ct Max	36	--	36	--	--	--	--

Disabled

**Step Name:** RT  
**Profile Type:** Hold **Cycles:** 1

Type	Time (s)	Temp (°C)	Detect
--	900	45	No

**Step Name:** Denaturation  
**Profile Type:** Hold **Cycles:** 1

Type	Time (s)	Temp (°C)	Detect
--	120	99	No

**Step Name:** PCR  
**Profile Type:** 3 - Temperature **Cycles:** 40

Type	Time (s)	Temp (°C)	Detect
--	10	99	No
--	10.6	68	No
--	13	72	Yes

	CoVN2			
CoVN2 - PCR	✓	✓	✗	✗
RNase P - PCR	✓	✗	✓	✗
Results	+ POS	+ POS	- NEG	! UNR

	CoVN1			
CoVN1 - PCR	✓	✓	✗	✗
RNase P - PCR	✓	✗	✓	✗
Results	+ POS	+ POS	- NEG	! UNR



Performed By: \_\_\_\_\_

Date: \_\_\_\_\_

Printed: 2020-06-26 10:08:53

Reviewed By: \_\_\_\_\_

Date: \_\_\_\_\_

Page: 1/1